

ARCHIPLAN 3D

INSTALLATION INSTRUCTIONS

01 | HOW CAN I UPDATE MY COMPUTER TO THE LATEST ARCHIPLAN VERSION?

Before installing a new Archiplan version, we recommend uninstalling the previous release completely.

Don't worry, the procedure is not going to remove your patients and projects from your database.

02 | WINDOWS INSTRUCTIONS

First, check that your computer meets the minimum system requirements (attached);

Open Archiplan software and click on Settings icon->Show setting folder (for Windows: C:\Users\nome Utente PC\AppData\Roaming\RealGUIDEsweden-martina). The name of the settings folder is different if you have installed a partner version.

Close Archiplan;

Delete all files located in the “RealGUIDEsweden-martina”. Before you do this, please be sure you saved a copy of your custom Archiplan files, if you created any (read this article to learn how to save custom Archiplan files).

click on Windows Start icon, look for Archiplan software in the programs list and click Uninstall (do not use the Search tool).

run the installer of the new Archiplan version and follow the proposed installation wizard.

03 | MAC INSTRUCTIONS:

First, make sure your computer meets the minimum system requirements (attached);

Open Archiplan and click on the Settings icon -> Show Settings folder (Macintosh>Utenti>nome Utente MAC >config.> RealGUIDEsweden-martina). The name of the settings folder is different if you have installed a partner version.

close Archiplan;


delete only and exclusively the files contained in the “ RealGUIDEsweden-martina” folder. Before you do this, please be sure you saved a copy of your custom Archiplan files, if you created any (read this article to learn how to save custom Archiplan files);

search for the Archiplan software in the MAC Finder, APP section, select it and click Move to Trash.

run the installer of the new Archiplan version and follow the proposed installation wizard.

In case of partner versions, install the latest release made available by the distributor itself.

CAUTION: Patient data are contained in the “RealGUIDE50-DB” folder. Deleting it would result in the permanent loss of data in local memory.

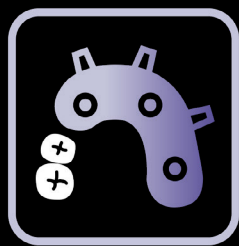

ARCHIPLAN 3D

MINIMUM HARDWARE REQUIREMENTS

MINIMUM HARDWARE REQUIREMENTS



PROCESSOR: Intel I5 or I7 (for laptops).
Quad Core recommended (for Workstations).
RAM: minimum 4 GB.
For big datasets 8 GB are suggested.
HARD DISK: 300 – 500 GB
GRAPHICS CARD: 3GB or higher vRAM recommended.
For higher performances Nvidia GTX or QUADRO line
SCREEN RESOLUTION: 1920x1080 (Full HD) for optimal visualization
OS: Windows 10, 11



MacBook Pro 15”
iMac series
OS: OS (12 Monterey, 13 Ventura)



iPad Pro
OS: iOS 16

In order to correctly manage the CLOUD platform a stable Internet connection is required.



WWW.SWEDEN-MARTINA.COM